

## 25mm SIDEARM PAD RAISERS

### FITTING GUIDELINES

**Part Numbers:-** L5-P22-F# (Suitable for most 8 & 9 series wheelchairs)



The enclosed Sidearm Pad Raisers are intended for use on wheelchairs with round tubular sidearm frames of 19mm (3/4") to 22mm (7/8") diameter, having Arm Pad Screw fixing centres of either 150mm or 200mm.

#### KIT CONTENTS:-

- 2 x Arm Pad Raiser Tubes,
- 8 x 22mm Saddle Washers
- 4 x M6 x 55mm Screws.

#### TOOLS REQUIRED:-

- Appropriate Screwdriver to remove existing Sidearm Pads.
- Pozidrive 3 Screwdriver for screws supplied in Kit

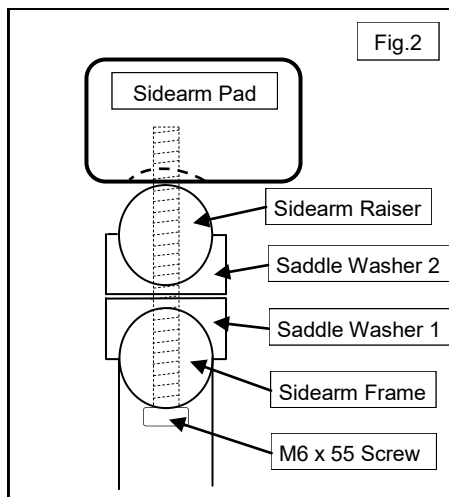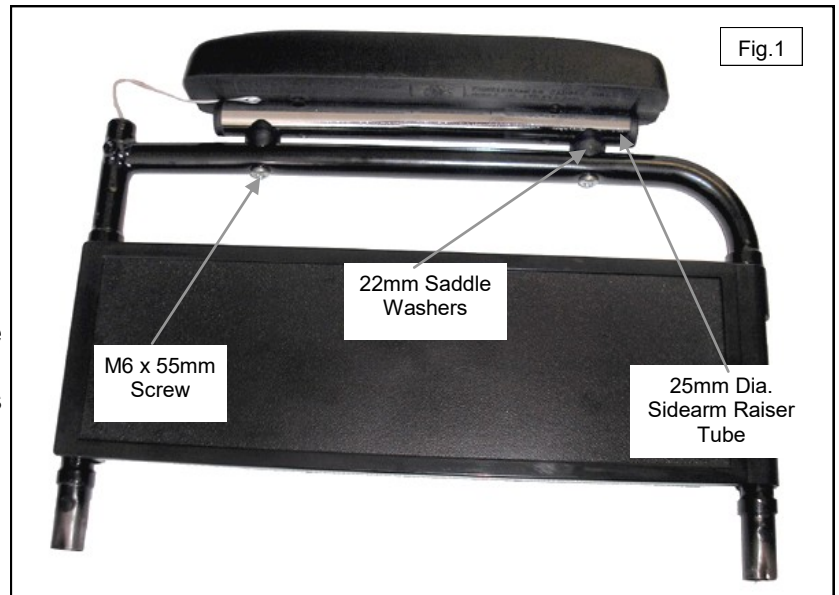

#### FITTING

- Apply wheelchair Parking Brakes.
- For ease of working and to avoid any possibility of injury, the wheelchair User should vacate the wheelchair whilst these accessories are being fitted.
- Disconnect and remove both Sidearm Pads from their frames.
- Taking one Sidearm Frame at a time, insert one of the M6 x 55mm Screws supplied, upwards through the forward Pad mounting hole. **Refer to Diag.2**
- Install onto the Screw, Saddle Washer 1, Saddle washer 2, then the Sidearm Raiser.
- Position Sidearm Pad over the Screw and engage Screw loosely.
- Repeat the above with the second Screw and finally tighten both Screws sufficient to prevent any movement.
- Carry the same procedure with the opposite Sidearm.

#### NOTE:

As with all wheelchair components, Sidearm Pads should be checked periodically for security.