



Thompson House, Unit 10, Styles Close  
Sittingbourne, Kent ME18 6DN  
Tel: 01795 477280  
E-mail: [sales@rms-kent.co.uk](mailto:sales@rms-kent.co.uk)  
[www.ineedawheelchair.co.uk](http://www.ineedawheelchair.co.uk)



## **ARMREST PADS**

### **FITTING INSTRUCTIONS**

Armrest Pads are manufactured to customer's order specifications and, unless otherwise specified, are supplied as a direct replacement for, or as an alternative to, the original equipment Armrest Pad. The method of attachment will therefore be as the original.

This would normally be by means of two M5 or M6 x 30mm Pan-head screws (supplied dependant on application), through the sidearm frame into "T" type Nutserts built into the base of the Pad.

To ensure correct fitting and support, this device should be only be fitted by a suitably qualified person.

#### **FITTING**

- Remove screws securing existing Armrest Pad to wheelchair sidearm frame.
- Clean sidearm frame.
- Locate replacement Pad in position on sidearm frame, aligning threaded holes in underside of Pad with mounting holes in sidearm frame.
- Apply a small amount of Loctite 243 or similar threadlock to threads of screws supplied.
- Insert screws upwards through sidearm frame into Armrest Pad and tighten to hand tight using a Pozidrive size 2 screwdriver.

\*\*\*\*\*

#### **USER / CARER INFORMATION**

- Check for Armrest Pad security on a regular basis.
- Upholstery should be checked on a regular basis and any damage reported to the appropriate authority for replacement as soon as possible.
- To ensure hygienic use, the vinyl covering of this Armrest Pad can be cleaned with warm soapy water with a little disinfectant added and wiped off with a clean dry cloth.

Should you require further information or advice regarding this, or any other products manufactured by RMS Ltd., please contact our Technical Helpline on 01634-578881