

RMS FOOT-STRAPS

(Covering Part Numbers FS003 S & L)
FITTING GUIDELINES


NOTES:

FS003 Foot-Straps are intended for use as either Ankle or Toe Straps or both and are suitable for use with Footplates or Footrest Panels having slots or holes to take 25mm nylon webbing.

(See examples below)

The overall length of the Short Foot-Strap (FS 003S) is 450mm and suitable for circumference range of 300 to 370mm.

The overall length of the Long Foot-Strap (FS 003L) is 500mm and suitable for circumference range of 350 to 420mm.

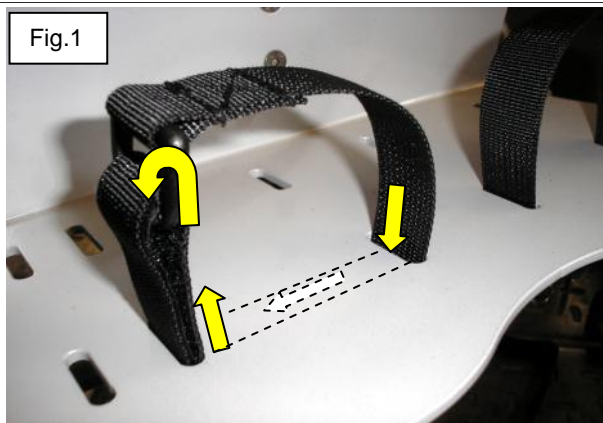

FITTING (Refer to Figs. 1 & 2)

- With the Velcro panels of the Foot-Straps facing towards the centre of the wheelchair, feed the free end of each Foot-Strap downwards through the appropriate slot in the Foot Plate/Panel.

- Route the Foot-Strap, across underneath the Foot Plate/Panel, to the opposite appropriate slot, then feed the Foot-Strap end upwards through the slot, (the Velcro panels should now be facing the outer edge of the Foot Plate/Panel).

- Feed the free end of each Foot-Strap through the "D" ring and secure by folding back onto the Velcro panels.

- Care should be taken not to over-tighten Foot-Straps when adjusting.



Foot-Strap shown on RMS Multi-Positional Footplate

NOTE:

RMS Foot-Straps may also so be used in conjunction with other RMS Postural Positioning devices. For further details of these, please refer to the current RMS Product and Accessory Catalogue or the RMS website: www.ineedawheelchair.co.uk

For technical assistance, please contact our Technical Help-line on 01795-477280