



June 2014

The Gill III Junior APPROVED ACCESSORIES

INSTALLATION GUIDELINES for EXTENDED FRAME PADDING

Intended for use with :-

OTTOBOCK – Kimba ACTIVATE – Tom Cross Maxi Hoggi – Bingo

The Extended Frame Padding is designed to be easily attached to the Buggy frame as shown below. Provision has been made, by means of zips on either side, to allow for the seat unit to be used in positions ranging from upright to fully reclined.

The Padding Kit is supplied in two pieces, a Left and Right side, which, after wrapping around the frame on each side as shown below, will finally join via Velcro panels in the centre behind the seat unit.



Wrap front wings of both Padded Panels around Backrest frames as shown and secure via the Velcro panels. Position to allow access to seat recline release catches and with the two closed tension adjustment zips facing outwards.

Overlap the Left-hand and Right-hand Panels at the rear of the Seat Unit and secure by means of the Velcro panels. Either one or both zips can be released to enable the Seat Unit to be reclined. Raise the zips as required when Seat Unit is raised to a more upright position.

WARNING

This accessory is not intended for use as a storage device or to hang objects from on the rear of the Buggy.

Should you require further information on this or any other product in the RMS range please contact our Technical Help-Line on 01795 477280

