

Gill 2 Wheelbase options

The Gill 2 seating system has been developed to fit into either an Invacare Action 3NG wheelchair or a Southwest Seating Neo tilting wheelchair. As a complete package, RMS can provide the Gill 2 fitted on either of the above wheelbases which are available from stock.

Action 3 self propelling manual wheelchair

- Compact adjustable wheelbase
- Suitable for transportation
- Indoor & outdoor use
- Quick release wheels
- Swingaway footrests
- Anti-tip levers
- Black frame



Neo tilting manual wheelchair


- Tilting wheelbase
- Suitable for transportation
- Indoor & outdoor use
- Height adjustable push handles
- Hub brakes
- Height adjustable armrests
- Removable footplate hangers
- Angle and depth adjustable footplates



Technical Specification

Standard Gill 2 consists of:

- One piece modular seating system including interface
- Growth adjustment in seat depth, backrest height, hip & thoracic widths
- Type D headrest
- Swing away thoracic supports
- Ramped seat cushion
- Full length adjustable pelvic supports
- Padded lap belt
- Machine washable covers
- A full range of accessories is available from the RMS Accessory Catalogue

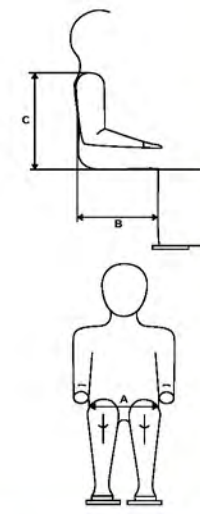

Gill 2 Seating System Size Guide		Size 1 GILL2-1-01-00	Size 2 GILL2-2-01-00	Size 3 GILL2-3-01-00	Size 4 GILL2-4-01-00
Pelvic width min/max (A)		200mm/275mm	250mm/325mm	315mm/375mm	365mm/425mm
Seat depth min/max (B)		225mm/300mm	300mm/375mm	360mm/435mm	430mm/505mm
Backrest height min (C)		355mm/430mm	390mm/465mm	450mm/525mm	500mm/575mm
Thoracic pad sizes H x L		78mm x 113mm	88mm x 138mm	98mm x 168mm	108mm x 188mm
Pelvic pad size H x L		75mm x 275mm	100mm x 325mm	100mm x 400mm	100mm x 475mm
Ramp length		145mm	170mm	180mm	190mm
Weight capacity for transport		50Kg	50Kg	50Kg	50Kg
Weight capacity		80Kg	80Kg	80Kg	80Kg
Seat unit weight		6Kg	7.8Kg	9.3Kg	10.9Kg
WHEELCHAIR OPTIONS	Self-propelling wheelchair	Action3 NG Junior 13 x 11	Action3 NG Junior 15 x 13	Action3 NG 17 x 16	Action3 NG 19 x 18
	Tilting wheelchair c/w Arm & Foot rests	Not Available	SW Seating Neo	SW Seating Neo	SW Seating Neo

Transportation

Successfully crash tested to ISO16840 part 4



Upholstery

There are five colour options available:

- Black/Red edging
- Black/Purple edging
- Black/Navy Blue edging
- Black/Royal Blue edging
- Black/Black edging

Machine washable covers



2 Seating System



RMS Rehabilitation Manufacturing Services Limited

Medway House, 277 Gillingham Road, Gillingham, Kent, ME7 4QX
 Tel: +44 (0)1634 578881 Fax: +44 (0)1634 581456
 Email: sales@rms-kent.co.uk www.ineedawheelchair.co.uk
 Rehabilitation Engineering For a Caring World

RMS Rehabilitation Manufacturing Services Limited



British design and quality



2 Seating System

The Gill 2 seating system is designed to meet the needs of wheelchair users unable to maintain a symmetrical and unsupported sitting position.

The Gill 2 is manufactured in four standard sizes for use in either a fixed or tilting manual wheelchair. The Gill 2 is fully adjustable allowing the seating system to grow with the individual, thereby, continuing to meet postural needs, whilst also ensuring a cost effective seating solution.

Thoracic Supports

Many wheelchair users are able to maintain an upright sitting posture if given assistance. The Gill 2 lateral supports maintain trunk support and alignment; they also swing away in order to promote ease of transfer.

Pelvic Support Guides

It is important that any seating system reduces the tendency of the pelvis to move obliquely or to rotate, which can lead to wind sweeping of the lower limbs. The Gill 2 pelvic supports guide the user's seated position into a neutral lateral tilt of the pelvis, preventing it from adopting an oblique or rotated position.

Foot Support

The Gill 2 uses the footplates from the host wheelchair.

Gill 2 fitted to Action 3 wheelchair



Headrest

The Gill 2 has an adjustable headrest which enables the user's head to be supported appropriately, even if the head cannot be supported conventionally in the midline.



Backrest

The Backrest cushion, when used in combination with the seat cushion, promotes stability to improve comfort and function. The Backrest cushion is fully adjustable in height and angle to maintain correct posture.



Seat Cushion

The Gill 2 cushion is designed to stabilise and anchor the pelvis which reduces the tendency to sit in a poor posture. The 15° ramped seat cushion captures and stabilises the ischial tuberosities and reduces the tendency to sit in a posterior pelvic tilted position.



Adjustability & Size

The Gill 2 is available in four sizes to provide the optimum sized seating system. The Gill 2 is fully adjustable for all widths, heights and depths; this can easily be achieved in clinic with no additional parts required.



Gill 2 fitted to Neo manual wheelchair