

Otto Bock®

QUALITY FOR LIFE

## Hubfix II: Solid Support for Bathing



Now the Hubfix Bathing Aid is even more practical. Telescoping side panels allow the use of several bathing supports, such as the Otto Bock Flipper or Robby, which can be set up in various sitting and lying positions. The panel surface is enclosed on the bathtub side—providing even greater safety for everyday use.

Simple and practical operation: the lowering and raising of the Hubfix is operated by a release knob via a gas compression spring. No additional water or power supply connections are needed.

The Hubfix II makes bathing easier on the patient and the caregiver's back. Four large suction cups provide Hubfix II with a solid grip so that it can be quickly and easily removed after use. Hubfix II: practical support for proper bathing.

The medical aid prescription number has been applied for.


Otto Bock HealthCare GmbH

Max-Näder-Straße 15 · D-37115 Duderstadt · Phone +49 5527 848-1461 · Fax +49 5527 848-1460 · reha@ottobock.de · www.ottobock.com

# Hubfix II



Flipper attached to the Hubfix II

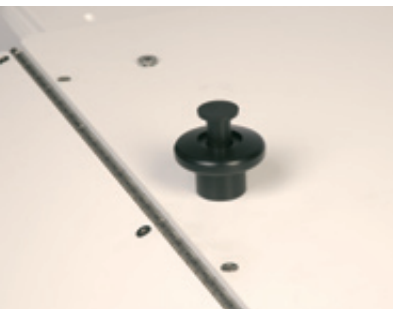

Release knob for gas compression spring



Telescoping cover panel, attachment



Four large suction cups provide for a solid grip

## Hubfix II in detail

- mechanical lifting aid for use in bathtubs
- no additional water or power supply connections are necessary
- telescoping support surface for attaching bathing supports such as the Flipper or Robby
- bathing supports are fastened with clamps
- lateral “wings“ provide greater safety
- gas compression spring
- four large suction cups
- quick and easy set-up/dismantling in the bathtub
- can be set up to exit bathtub on right or left side

## Technical Data

Overall length:	99 cm to 122 cm
Overall width:	41 cm to 71 cm
Height, folded:	10 cm
Maximum height:	41 cm
Maximum load capacity:	30 kgs (66 lbs)
Total weight:	12.5 kgs (27.5 lbs)



**RMS**

**Otto Bock®**

QUALITY FOR LIFE

RMS Limited, Thompson House  
Unit 10, Styles Close, Sittingbourne  
Kent ME10 3BF  
T: 01795 477280 E: sales@rms-kent.co.uk