


RMS Sidearm Raisers

FITTING GUIDELINES

Covering Part Numbers:

L5-P01-F3A, L5-P01-F3B, L5-P01-F3C and L5-P02-F3

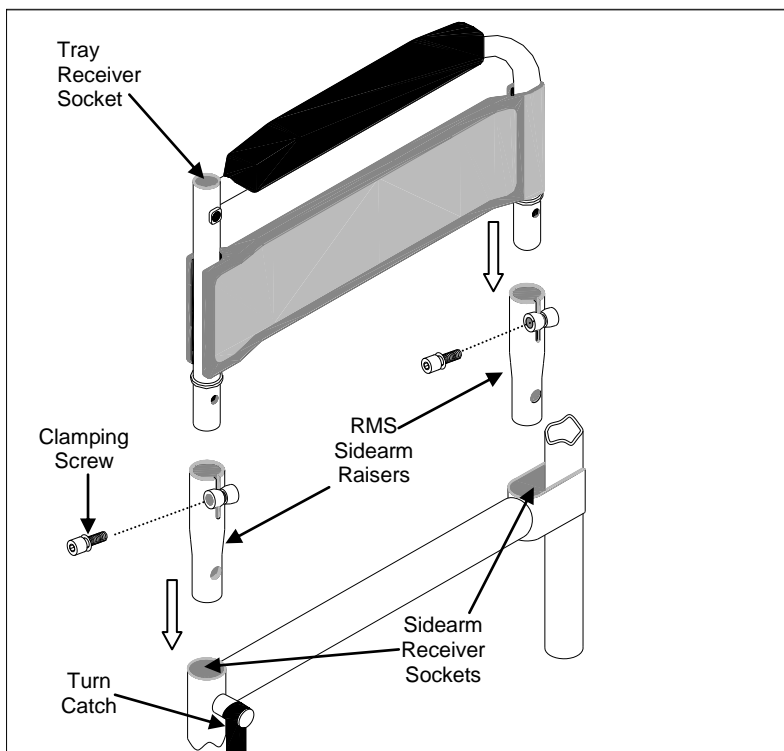

NOTES:

- It is recommended that after installation, the Sidearm Raisers are left permanently attached to the sidearm frames. This allows the sidearms to be locked in place by means of the original Turn Catches.
- RMS Sidearm Raisers are handed and should always be fitted with the clamping bushes facing away from the user

TOOLS REQUIRED:

M5 Hexagon Key

FITTING:

- Remove sidearms from wheelchair and insert Sidearm Raisers into the sidearm receiver sockets at front and rear, (ensure Raisers are secured by the sidearm turn catches).
- Refit sidearms into Raisers (as diagram) and tighten Front clamping screws, using an M5 Hexagon Key, sufficient to prevent any movement.
- Remove sidearms complete with rear Raisers and turn through 180°. Insert previous rear Raisers into front sockets and secure in place by the Turn Catches. Re-insert sidearms and tighten remaining clamping screws.

USER/CARER INSTRUCTIONS:

Sidearm Raiser clamping screws should be checked for security on a regular basis by a competent person.