

RMS Sidearm Raisers

FITTING GUIDELINES

Covering Part Numbers:


L5-P01-F3A, L5-P01-F3B, L5-P01-F3C and L5-P02-F3

NOTES:

- It is recommended that installation should be carried out by a suitably qualified person.
- The Sidearm Raisers should be left permanently attached to the sidearm frames. This allows the sidearms to be locked in place by means of the original Turn Catches.
- RMS Sidearm Raisers are handed and should always be fitted with the clamping bushes facing away from the user



TOOLS REQUIRED: M5 Hexagon Key

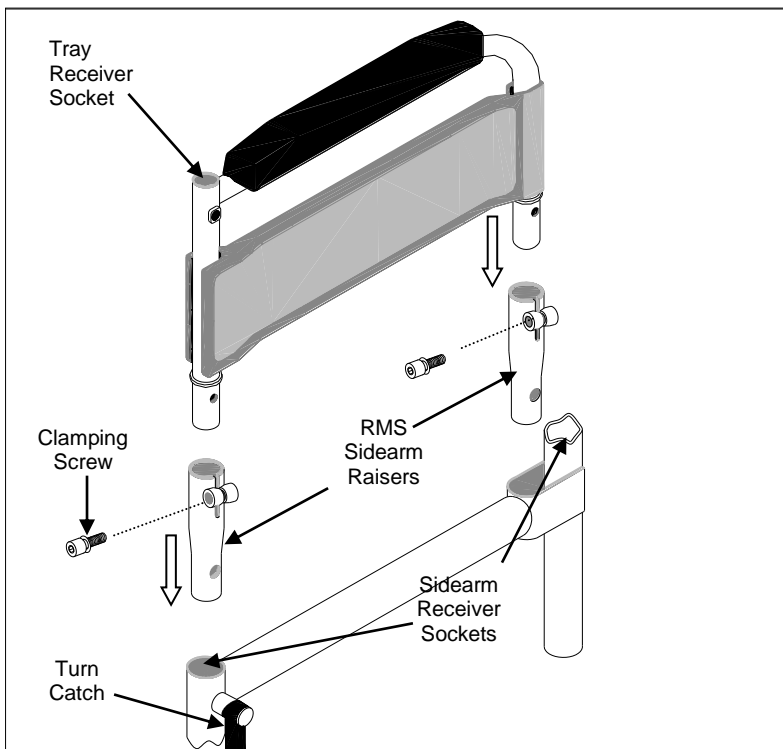

FITTING:

- Remove sidearms from wheelchair and insert Sidearm Raisers into the sidearm receiver sockets at front and rear, (ensure Raisers are secured by the sidearm turn catches).
- Refit sidearms into Raisers (as diagram) and tighten Front clamping screws, using an M5 Hexagon Key, sufficient to prevent any movement.
- Remove sidearms complete with rear Raisers and turn through 180°. Insert previous rear Raisers into front sockets and secure in place by the Turn Catches. Re-insert sidearms and tighten remaining clamping screws.

USER/CARER INSTRUCTIONS:

- Sidearm Raiser clamping screws should be checked for security on a regular basis by a competent person. Any damage should be reported to an appropriate authority.
- Caution: the wheelchair skirt guards will be raised above seat level. Please keep clothing and hands away from wheelchair wheels and moving parts.