

RMS Leg Abductor Straps

For Part Numbers LAS-001, LAS-002 and LAS-003



RMS Leg Abductor Straps are intended for use as a method of holding each inner thigh and providing resistance to pelvic obliquity and rotation.

The free ends should be attached in a suitable position to the wheelchair frame using the clamps supplied.

After attaching to the wheelchair frame, and with the user seated, route the lower strap under the user's thigh and the upper strap over the thigh. Connect the strap at the pull-tight buckle.

Adjustment is made by pulling as necessary on the "D" ring. To avoid circulation problems, care should be taken not to over tighten these straps, when adjusting using the pull tight buckle.

Should you require further information regarding these or any other products in the RMS range, please contact the RMS Technical Help-line on 01795 477280.